

---

14-5-2018



# WESSELING BV

---

Quality since 1958

## OPTIMUS KOMBI

USER'S MANUAL AND MAINTENANCE GUIDE



CE

**CONTENTS**

**1. GENERAL INFORMATION ..... 2**  
 MANUFACTURER ..... 2  
 EQUIPMENT INTENDED USE AND DESCRIPTION ..... 2  
 STORAGE ..... 2  
 SYMBOLS ..... 2

**2. SPECIAL WARNINGS AND USE RESTRICTIONS ..... 3**

**3. USE ..... 3**  
 STARTING UP ..... 3  
 SETTING UP AND ADJUSTMENTS ..... 3  
 TRANSPORTATION AND BRAKES ..... 4  
 FITTING AND USING THE ACCESSORIES-OPTIONAL EXTRAS: ..... 4

**4. CLEANING and CONSERVATION .....6**

**5. MAINTENANCE ..... 6**  
 SPARE PARTS: .....6

**6. TROUBLESHOOTING .....7**

**7. TECHNICAL DATA SHEET .....8**

**8. WARRANTY .....9**

**9. SERVICE RECORD .....9**

**1. GENERAL INFORMATION**

**EQUIPMENT INTENDED USE AND DESCRIPTION**




The Optimus Kombi is intended for examining and treating patients. With the additional options this table can be used for treatment and examination. The entire frame is epoxy powder coated and the top is upholstered and covered in fireproof synthetic leather. It's equipped with a high quality actuator for height adjustment. Head section as well as back section can be inclined with the help of gas springs. The gas springs are all adjusted by handles that are easy to access on the side of the table or on the leg section. The brakes and lock are centrally controlled by means of a foot pedal that is easy to activate.

**STORAGE**




When storing, the following conditions must be met:  
 1. Relative humidity 10% / 90 % - temperature -10°C / +50°C

**SYMBOLS**

The various symbols appearing in this manual and on the equipment itself are meant to draw the user's attention on important safety warnings and useful information. Please observe all warning, caution and hazard notices.

	<b>WARNING</b> Warning messages are used to advise users/patients about any risk of injury or danger if the specified procedures and conditions are not carefully observed during operation or maintenance activities.
	<b>ELECTRIC SHOCK HAZARD</b> This warning symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in a risk of electric shock
	<b>CAUTION</b> Caution symbols are used to advise users about an operation or maintenance procedure, best practice or condition that may damage the equipment if not duly complied with.

## 2. SPECIAL WARNINGS AND USE RESTRICTIONS

	<p><b>Equipment must not be used:</b></p> <ul style="list-style-type: none"> <li>• Without knowing the user's manual.</li> <li>• When the safety load exceeds the stated capacity.</li> <li>• Without the patient being monitored.</li> <li>• When the patient and/or equipment is not stable enough.</li> <li>• When table is not on its feet when stopped, during treatment, or when patient is moved to/from other equipment.</li> <li>• When the original product has been tampered with and/or modified from its original configuration.</li> <li>• When the controls and/or components are even only partially faulty or defective.</li> <li>• When incorrectly configured or when you think it may cause damage to a patient or user.</li> <li>• Do not allow any body parts to enter the areas with moving components and/or controls.</li> </ul>
	<ul style="list-style-type: none"> <li>• Do not connect to an electric system that does not comply with the current safety regulations.</li> <li>• Never spill liquid of any kind on the electric equipment.</li> <li>• Always unplug the mains power supply before working on any electrical parts or to move the equipment.</li> <li>• Do not damage the electric cables: avoid crushing; do not pull out the cable to disconnect, etc.</li> <li>• Electric cables should be checked regularly to see if they are damaged. In case of doubt, have them replaced.</li> </ul>
	<ul style="list-style-type: none"> <li>• Foam rubber and plastic components may burn if they come in contact with naked flames.</li> <li>• Corrosive and/or abrasive detergents may damage the covers and paintwork.</li> <li>• The equipment is not suitable for use outdoors and/or in surfaces that are not smooth enough.</li> <li>• UV and heat therapy lamps may damage the upholstery covering.</li> </ul>
<b>!!!!!!!!!!!! NEVER LEAVE THE TABLE UNATTENDED WHEN THERE IS A PATIENT ON IT !!!!!!!!!!!!!</b>	






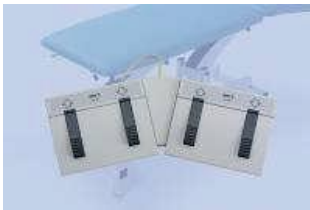
## 3. USE


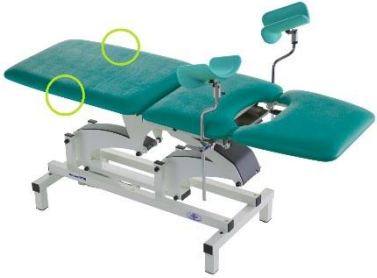


### STARTING UP

For packaging reasons the couch is supplied covered by protective means that are to be removed, avoiding direct contact with cutters that could damage it. Any damage in transit must be reported to the transporter and seller immediately, without using the equipment for safety reasons.
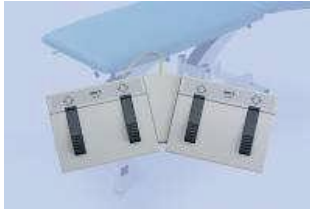
### SETTING UP AND ADJUSTMENTS


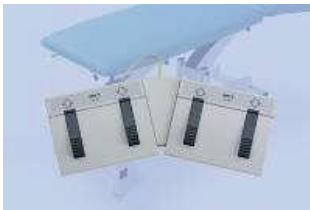
The table has two electric actuators: one for the height adjustment and one for the tilt of the sitting section. The head section and the back section are both fitted with gas springs.

ADJUSTMENT/FUNCTION	CONTROL	PROCEDURE
		Table is standard equipped with a 24v foot switch or hand switch.
		Push the relevant symbol to adjust the height of the entire table.
		Push the relevant symbol to adjust the height of the sitting section.

		<p>To adjust the back sections pull the control handle, and support the section as it moves. Release the handle to secure the position. The control handles are on both sides of the table.</p> <div style="border: 1px solid black; padding: 5px; text-align: center;"> <b>WARNING</b> </div> <ul style="list-style-type: none"> <li>• With the patient’s weight on the section, activate the control handle only when you are sure that the entire load is supported.</li> <li>• Movement of the adjustable sections may cause pinching in the vicinity of the hinges.</li> <li>• Do not sit on the back section</li> </ul>
		<p>To adjust the leg section pull the control handle, and support the section as it moves. Release the handle to secure the position.</p> <div style="border: 1px solid black; padding: 5px; text-align: center;"> <b>WARNING</b> </div> <ul style="list-style-type: none"> <li>• With the patient’s weight on the section, activate the control handle only when you are sure that the entire load is supported.</li> <li>• Movement of the adjustable sections may cause pinching in the vicinity of the hinges.</li> <li>• Do not sit on the back section</li> </ul>

**TRANSPORTATION AND BRAKES**

The table has four swivel wheels to allow movements in tight spaces. At the back-side of the table is the stirrup bar of the wheel-lift system. By pushing this stirrup bar downwards the table becomes mobile. Lift the stirrup bar upwards again after reaching the required position so the castors will be retracted. Never use the table if the castors are not fully retracted! The 4 feet can be adjusted individually when the floor is not even.

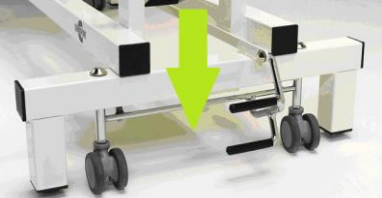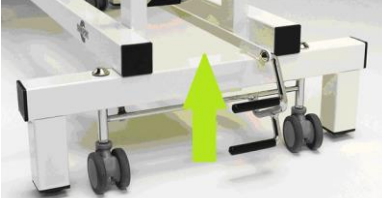

MOVING THE TABLE	PUT THE TABLE ON ITS FEET	PROCEDURE
		<ul style="list-style-type: none"> <li>• To move the table use the foot to push down the bar till it stops.</li> <li>• To put the table on its feet use the foot to push up the bar until all the feet are on the floor.</li> </ul>





**WARNING**

- Putting the table on its feet is essential when transferring and treating the patient.

**FITTING AND USING THE ACCESSORIES-OPTIONAL EXTRAS:**


The table’s use can be extended by means of the accessories - optional extras

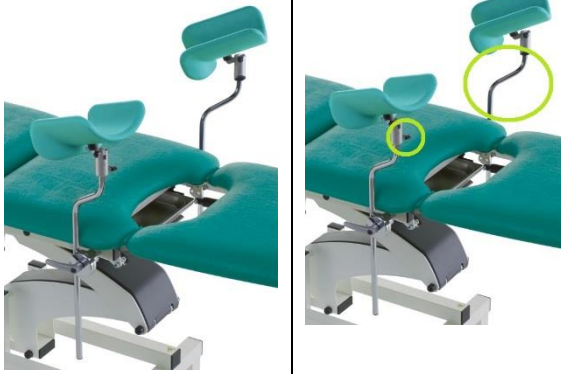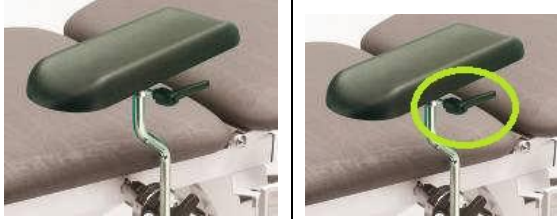ACCESSORY – OPTIONAL EXTRA	CONTROL	PROCEDURE
<p style="text-align: center;">Adapters</p> 		<ul style="list-style-type: none"> <li>• Loosen the lever to open the adaptor.</li> <li>• Put the knee support in the adaptor</li> <li>• Put the knee support at the desired height and position and fasten the lever.</li> </ul>

<p>Knee supports</p> 	<p>The knee supports are adjustable in height and width. This option can only be fitted in combination with the adaptors</p> <ul style="list-style-type: none"> <li>• To adjust the height: loosen the lever of the adapter and put the knee-support at the desired height. Than fasten the lever again to keep the knee-support in place.</li> <li>• To adjust the width: loosen the lever of the adapter and put the knee-support at the desired width by twisting the support. Than fasten the lever again to keep the knee-support in place.</li> <li>• The black small lever on the support is to keep it in place.</li> </ul>
<p>Arm support</p> 	<p>The arm support is adjustable in height and width. This option can only be fitted in combination with the adaptors</p> <ul style="list-style-type: none"> <li>• To adjust the height: loosen the lever of the adapter and put the arm-support at the desired height. Than fasten the lever again to keep the arm-support in place.</li> <li>• To adjust the width: loosen the lever of the adapter and put the arm-support at the desired width by twisting the support. Than fasten the lever again to keep the arm-support in place.</li> </ul>
<p>Drip pole</p> 	<p>The drip is only adjustable in height This option can only be fitted in combination with the adaptors</p> <ul style="list-style-type: none"> <li>• To adjust the height: loosen the lever of the adapter and put the drip pole at the desired height. Than fasten the lever again to keep the drip pole in place.</li> </ul>
<p>Papperrolholder</p> 	<p>Papperrolholder is fitted on the back of the Back-section. The width adjustable to allow use of different widths of paper.</p>

## 4. CLEANING and CONSERVATION

For durable joy with the leather material, please clean regularly. For cleaning use a cloth or soft brush dampened using water and a neutral detergent. Any stains on the upholstery should be promptly removed with colourless ethyl alcohol diluted in 20% water. Rinse well.

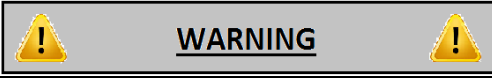

- ENSURE THE COUCH IS PERFECTLY CLEAN BY DISINFECTING THE SYNTHETIC LEATHER AND APPLYING A SINGLE-USE PAPER SHEET AT EVERY USE.
- HARSH DETERGENT PRODUCTS MAY DAMAGE THE SURFACES.
- DO NOT USE STEAM JETS, WASHING TUNNELS, WATER JET CLEANERS OR HIGH PRESSURE WASHERS

## 5. MAINTENANCE

Maintenance operations must be carried out by technicians who are familiar with the contents of this instruction booklet. Check the equipment is safe after any instances of "rough handling" (e.g. fall, bumps, etc.) and any time you are in doubt about safety.


### ROUTINE MAINTENANCE

Routine maintenance operations must be carried out with the frequency indicated in the table.

FREQUENCY	ACTIVITY
Every 6 months or at least once a year	General equipment cleaning.
	Visual and functional inspection of the equipment and related hydraulic and manual controls.
	Frame and mechanical parts: Check screws and bolts are tight; state of wear of bushings and pins, presence of tears and deformations or damages.
	Oil leaks from the piston and/or gas springs must only be in the form of minor lubrication residue.
	Check that the adjustable sections are stable and do not give way when the control lever is released.
	Brake control: check regular functioning of the activating device and the efficiency of the brakes.
	Wheels: these must be securely fixed and in good condition, rolling smoothly and with their treads free of impediments.
During use and operation, make sure there are no unusual sounds, vibrations or oscillations that may indicate actuator and/or mechanism malfunctions.	
At the end of the activities above, complete the inspection performing a lift cycle with the stated maximum safe load.	

### NON-ROUTINE MAINTENANCE

For safety reasons and to avoid invalidating the guarantee, no component may be opened or handled for repair. Any faults on the hydraulic piston, the gas springs and the wheel system must be diagnosed by Wesseling BV, who will organize the most suitable action.

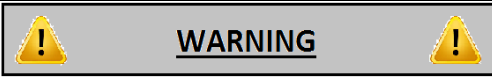

- IF SAFETY SEEMS TO BE AT RISK, DISABLE THE EQUIPMENT IMMEDIATELY PREVENTING ANY POSSIBILITY OF USE UNTIL SUCH RISK HAS BEEN REMOVED.
- PLEASE RECORD ANY DEFECTS/DAMAGES/SERVICES THAT ARE IMPORTANT FOR THE SAFETY OF THE EQUIPMENT IN THE APPROPRIATE FORM HEREWITH ENCLOSED.

### SPARE PARTS

Only use original spare parts from Wesseling. Please contact Wesseling BV for the parts you might need.

## 6. TROUBLESHOOTING

DIRECT E-MAIL FOR TECHNICAL OR USERS ASSISTANCE:

[info@wesseling-bv.com](mailto:info@wesseling-bv.com)

If you rather want to speak to one of us on the phone you can call: +31-20 497 5183



**Please note:**

- Before contacting for service or help, please identify the "SERIAL NUMBER" in the product label attached to the equipment (see example above), or the transport document/invoice number. This will speed things up and improve service quality.

The equipment was tested in very stringent conditions. During testing no specific breakages or faults occurred. If the equipment fails to operate, please follow the troubleshooting process below.

Issue	Cause	Solution
Adjustment of the head-/back section doesn't work or the position is not maintained	Gas spring faulty or incorrectly set	Contact Wesseling BV
	Transmission cable damaged or broken	
The actuator does not work	The attached switch is broken or not connected properly	Contact Wesseling BV
	Power cable is damaged or not connected properly	
	Actuator is damaged or not connected properly	

## 7. TECHNICAL DATA SHEET

<b>Product Name</b>	Optimus Kombi Electric, with wheels
<b>Product Code</b>	OPTKOMEMW00
<b>Safe working load</b>	200kg
<b>Max patient weight</b>	175kg
<b>Height adjustment:</b>	60 to 110 cm with a speed of 6,5mm/s - 6.000N, stroke 200mm
<b>Adjustment system</b>	Electric actuator
<b>Adjusting electrics</b>	24v foot- or hand switch
<b>Sections adjusted by actuator</b>	2
<b>Sections adjusted by gas spring</b>	2
<b>Leg section</b>	-85° to +25°
<b>Sitting section</b>	-5° to +15°
<b>Back section</b>	-15° to +75°
<b>Adjusting gas spring back section</b>	By levers that are on both sides of the table
<b>Adjusting gas spring head section</b>	By the lever that is under the leg section itself
<b>Wheels</b>	Ø75mm
<b>Braking system</b>	Centralized control on the back of the table
<b>Equipment weight</b>	± 82kg
<b>Length</b>	190cm
<b>Width</b>	63cm
<b>Upholstery thickness</b>	± 40mm
<b>Filling</b>	Polyether SG 90/30
<b>Semi Leather</b>	Artificial leather
<b>Principal dimensions and adjustment of the sections</b>	<p>The technical drawing illustrates the principal dimensions and adjustment ranges of the Optimus Kombi Electric table. It consists of two perspective views of the table's seating area, showing the backrest, seat, and leg sections. The top view shows the table in a reclined position with the backrest tilted at +75°, the seat tilted at +35°, and the leg tilted at -85°. Dimensions are provided: a total length of 190, a width of 63, and a depth of 47. The bottom view shows the table in a flat position with the backrest tilted at +5° and the seat tilted at -15°.</p>



**8. WARRANT**

**GENERAL AFTER-SALES SERVICE CONDITIONS FOR WESSELING ITEMS**

The warranty has a 24-month validity from the date of delivery.  
 The warranty does not cover consumable items, rechargeable batteries, and generally all the material subject to wear, breakdowns caused by knocks, falls, misuse or improper use, accidental events and transport damage. This warranty shall be automatically invalidated if the equipment is tampered with.  
 Repair of the equipment will be carried out in relation to the breakdown stated.  
 An estimate must be specifically requested.  
 Payment is cash on delivery, unless other agreements are made.  
 For any dispute, the sole competent court shall be the court of Amsterdam

**WARRANTY REPAIRS**

Warranty repairs must be expressly requested.  
 Repairs under warranty shall be carried out at our workshop and upon authorization  
 The return shipment for products sent in their original package will be made free of carriage charges.  
 A fiscal document (bill of sale, purchase invoice or cash receipt) is required to make a warranty claim; the purchase date indicated must fall within the warranty time period.  
 If, upon inspection, the product appears to be fully functioning, troubleshooting expenses shall still be charged to the client.

**OUT OF WARRANTY REPAIRS**

For out-of-warranty repairs carried out at our workshops upon authorization, transportation charges shall be paid by the customer.  
 The warranty of the parts replaced during the repair operations is 12 months from the moment the equipment is collected.  
 If, upon inspection, the product appears to be fully functioning, troubleshooting expenses shall still be charged to the client.

**REPAIRS MADE AT HOME**

In the event repairs are made at the customer's home, a written request is required indicating the complete details of the person making the request, the type of machine and the breakdown.  
 The cost per kilometer for on-site technician visits will be agreed upon in relation to the urgency of the customer.  
 In the event the machine is under warranty, the customer will have to pay only the expenses incurred for the technician to reach the customer's house.  
 Time is calculated from the moment the technician leaves our workshop until he/she returns. The time of his/her return will be estimated based on the time required to get there.

**SALE OF SPARE PARTS AND CONSUMABLE ITEMS**

The orders must be in writing and specify the customer, the item and the shipping method as well as the precise fiscal data of the customer.  
 The order processing time can vary depending on the availability of the material.  
 Returned spare parts will not be accepted.  
 Payment will be cash on delivery, unless other agreements are made.

**9. SERVICE RECORD**

EQUIPMENT:		DATE OF FIRST INSTALLATION:		SERIAL NUMBER:	
DATE OF SERVICE	OPERATIONS CARRIED OUT	TECHNICIAN	SIGNATURE	NEXT DUE DATE	